

Curan Lady





Step 1

Wash your hands thoroughly with soap and water and dry them. Make yourself comfortable. If there is a possibility you might be able to urinate, always try before you use a catheter.

Step 2



Wash around the opening of the urethra using an alcoholfree wet wipe or mild soap and water. Wash from front to back to reduce the risk of infection.

Break the transparent label on

the catheter by twisting the

easy-grip lid in the opposite

indicated by the arrows. If

required, attach a urine bag

to the drainage end of Curan

direction to the base as

Lady using the universal



Step 5

Find a position where you are most comfortable and can locate the urethra. Here are some of the positions that you may find helpful to perform ISC.

Lying propped up on the bed.

Sitting over the toilet.

Sitting in a wheelchair.

If you are doing ISC in a wheelchair, or lying propped up on the bed, it might be useful to attach a urine drainage bag to the catheter or have an appropriate receptacle to hand to collect urine.



Step 6

Spread the labia apart gently with the index finger and middle finger of one hand. You may need to pull slightly upwards to expose the urethra.

Step 7

With your other hand gently insert the catheter into your urethra as you have been shown, until it reaches the bladder and urine starts to flow.

Step 8

When the flow stops, ease out the catheter slowly to ensure that urine is no longer draining. If urine flow starts again, stop withdrawing the catheter. The bladder needs to be fully emptied to reduce the risk of infection.



Step 4

connector.

Step 3

Take hold of the catheter funnel end and pull the catheter from tube. Curan Lady is lubricated with a gel coating so there are no concerns about water spillage.



CliniMed®

For more information visit: clinimed.info/curan-booklet

Or call: 0800 585 125 Scan:



Step 9

Dispose of the catheter and the outer packaging in a waste bin. Do NOT flush it down the toilet. Wash your hands again with soap and water. If there is no immediate method of disposal available Curan Lady can be replaced into the tube and fully resealed to carry with you until a bin is located. Curan Lady is a single use catheter and should not be reused.





Curan Advantage Female



Step 1

Wash your hands thoroughly with soap and water and dry them. Make yourself comfortable. If there is a possibility you might be able to urinate, always try before you use a catheter.

Step 2

Wash around the opening of the urethra using an alcoholfree wet wipe or mild soap and water. Wash from front to back to reduce the risk of infection.



Step 3

Before opening the packaging, release the sterile water by lightly pinching or squeezing the integrated sachet until it bursts.



Step 4

Wait approximately 25 seconds for the sterile water to activate the hydrophilic coating. You can use the adhesive tab at the back to hang the packaging on a vertical surface whilst you prepare for catheterisation.





Step 6 Find a position where you are most comfortable and can locate the urethra. Here are some of the positions that you may find helpful to perform ISC.

Step 5

When you are ready to

catheterise, open the tab at

the front of the packaging by

pulling upwards, then take hold

of the catheter funnel end and

this stage, if required, attach a

urine bag to the drainage end

of Curan Advantage using the

the longer unisex catheter you

can use the Blue Grip® to hold

the catheter and help guide

insertion, without directly

touching the catheter itself.

universal connector. If using

Lying propped up on the bed.

Sitting over the toilet.

Sitting in a wheelchair.

If you are doing ISC in a wheelchair, or lying propped up on the bed, it might be useful to attach a urine drainage bag to the catheter or have an appropriate receptacle to hand to collect urine.







Step 7

Spread the labia apart gently with the index finger and middle finger of one hand. You may need to pull slightly upwards to expose the urethra.

Step 8

With your other hand gently insert the catheter into your urethra as you have been shown until it reaches the bladder and urine starts to flow

Step 9

When the flow stops, ease out the catheter slowly to ensure that urine is no longer draining. If urine flow starts again, stop withdrawing the catheter. The bladder needs to be fully emptied to reduce the risk of infection.

advantage

CliniMed®

CliniMed Ltd., a company registered in England number 01646927. Registered office: Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY. Tel: 01628 850 100 Email: enquiries @clinimed.co.uk or visit www.clinimed co.uk. Curan* and Blue Grip* are registered trademarks of Curan Medical BV, 7006 GE Doetinchem, The Netherland ComfortCoat* is a registered trademark of DSM Biomedical B.V., Koestraat 1, 6167 RA, Geleen, The Netherlands. CliniMed and SecuriCare® are registered trademarks of CliniMed (Holdings) Ltd.© CliniMed Ltd. 2024. PID 11604

For more information visit: clinimed.info/curan-booklet



Step 10

Dispose of the catheter and the outer packaging in a waste bin. Do NOT flush it down the toilet. Wash your hands again with soap and water. If there is no immediate method of disposal available, Curan Advantage can be replaced in the packaging and resealed using the adhesive tab to carry with you until a bin is located, empty out any water before your reseal the packaging. Curan Advantage is a single use catheter and should not be reused.

