



No Sting  
**Medical Adhesive Remover  
Spray**

**REF** 3501

**CliniMed**®

# Appeel<sup>®</sup> Advance No Sting

## Medical Adhesive Remover Spray

---

### Product Description

Appeel Advance No Sting Medical Adhesive Remover Spray contains a no sting healthcare-grade silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. Appeel Advance No Sting Medical Adhesive Remover helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

### Indications for use

Appeel Advance No Sting Medical Adhesive Remover Spray is indicated for use on intact skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

### Contraindications

Appeel Advance No Sting Medical Adhesive Remover Spray is not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

### Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Do not use near eyes, ears, nose, or mouth. Avoid direct inhalation or use in enclosed spaces. May not be suitable for use on babies under 30 days old.

### Instructions For Use

1. Skin should be clean and dry prior to application.
2. Spray Appeel Advance No Sting Medical Adhesive Remover Spray around the edge of the adhesive appliance to reduce skin trauma.
3. Wait a few seconds, then gently peel adhesive appliance away from the skin using additional spray to ease removal, if necessary.
4. Dries in seconds. Allow to evaporate completely before covering. Once dried Appeel Advance No Sting Medical Adhesive Remover Spray will not affect adhesion of further appliances.

# Appeel<sup>®</sup> Advance No Sting

## Medical Adhesive Remover Spray

### Warning















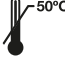



Flammable aerosol. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition source. Do not pierce or burn, even after use. Do not expose to temperatures above 50°C/122°F. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the can is damaged. Contact CliniMed in the event that the can is received damaged before use. Recycle empty cans in accordance with local regulations.

### Ingredients

Siloxanes

### Storage and Shelf Life

Store away from direct sunlight at ambient temperature. For shelf life, refer to the expiry date on the base of the can.

	Manufacturer		Date of Manufacture/Country of Manufacture		Flammable
	EC Representative		Date of Manufacture/Country of Manufacture		Recycling (Aluminium)
	Catalogue Number		Instructions for Use		Not Made with Natural Rubber Latex
	Batch Code		Caution/Warning		Brimful Volume
	Medical Device		Keep away from Sunlight		Upper Temperature Limit
	Unique Device Identifier		Single Patient Multiple Use		Use-by Date


Product Code	Description
3501	Appeel Advance No Sting Medical Adhesive Remover Spray

**CliniMed<sup>®</sup>**

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed<sup>®</sup> and Appeel<sup>®</sup> are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022  
PID 10909

