

No Sting
**Medical Adhesive Remover
Wipes**

REF 3505

CliniMed®

Appeel[®] No Sting

Medical Adhesive Remover Wipes

Product Description

Appeel No Sting Medical Adhesive Remover Wipes contain a no sting healthcare-grade silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. Appeel No Sting Medical Adhesive Remover helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

Indications for use

Appeel No Sting Medical Adhesive Remover Wipes are indicated for use on intact skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

Contraindications

Appeel No Sting Medical Adhesive Remover Wipes are not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Wipes are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of safely after use.

Instructions For Use

1. Skin should be clean and dry prior to application.
2. To open the sachet, tear along either notch across the top.
3. Remove the wipe from the sachet and gently wipe over the adhesive appliance to release it from the skin.
4. Gently lift and repeat as needed. Use additional wipes if required for larger adhesive appliances.
5. Dries in seconds. Allow Appeel to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.

Appeel[®] No Sting

Medical Adhesive Remover Wipes

Warning




















Appeel is highly flammable. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

Ingredients

Siloxanes.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.

	Manufacturer		Instructions for Use		Sachet Material Not Yet Recycled
	EC Representative		Date of Manufacture/ Country of Manufacture		Recycling (Paper and Cardboard)
	Catalogue Number				Do not use if packaging is damaged and consult instructions for use
	Batch Code		Caution/Warning		
	Medical Device		Flammable		Not Made with Natural Rubber Latex
	Unique Device Identifier		Keep away from Sunlight		Do Not Re-use
	Use-by Date		Keep Dry		Do Not Flush


Product Code	Description	No. Per Box
3505	Appeel No Sting Medical Adhesive Remover Wipe	30

CliniMed[®]

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed[®] and Appeel[®] are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd. Tower Business Centre, 2nd Floor,
Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022
PID 10914

