

Appeel® Sterile Medical Adhesive Remover Foam Applicator - 1ml

Product Description

Appeel Sterile No Sting Medical Adhesive Remover Foam Applicator is an applicator that contains a no sting healthcare-grade silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. Appeel Sterile No Sting Medical Adhesive Remover Foam Applicator helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

Indications for use

Appeel Sterile No Sting Medical Adhesive Remover Foam Applicator is indicated for use on intact and injured skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

Contraindications

Not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of safely after use.














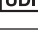


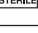




Instructions for use

1. Skin should be clean and dry prior to application.
2. Wear sterile gloves to maintain sterility when using this product.
3. To open the sachet, tear along either notch across the top.
4. Remove the foam applicator from the sachet and gently wipe over the adhesive appliance to release it from the skin.
5. Gently lift and repeat as needed. Use additional applicators if required for larger adhesive appliances.
6. Dries in seconds. Allow Appeel Sterile No Sting Medical Adhesive Remover Foam Applicator to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.

Warning: Appeel Sterile No Sting Medical Adhesive Remover Foam Applicator is highly flammable. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

Ingredients: Siloxanes

Storage and Shelf Life: Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.

	Manufacturer		Caution/Warning		Flammable
	EC Representative		Keep away from Sunlight		Sachet Material Not Yet Recycled
	Catalogue Number		Keep Dry		Do not use if packaging is damaged and consult instructions for use
	Batch Code		Single Sterile Barrier System		
	Medical Device		Single Sterile Barrier System with Protective Packaging Outside		Do Not Re-sterilise
	Unique Device Identifier			Do Not Re-use	
	Country of Manufacture		Sterilised by Irradiation		Recycling (Paper and Cardboard)
	Instructions for Use		Use-by Date		Not Made with Natural Rubber Latex

Product Code	Description	No. Per Box
3515	Appeel Sterile Medical Adhesive Remover Foam Applicator, 1ml	5

CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
 Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk
 CliniMed® and Appeel® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd, Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022
PID 10865


1639 0120