

# LBF® Sterile No Sting Barrier Film Foam Applicator - 1ml & 2ml

## Product Description

LBF Sterile No Sting Barrier Film Foam Applicator is an applicator that contains a no sting healthcare-grade silicone fluid that forms a transparent film on the skin and that acts as a protective barrier against damage from bodily fluids and waste and skin stripping caused by repeated application and/or removal of adhesives.

## Indications for use

LBF Sterile No Sting Barrier Film Foam Applicator is indicated for the protection of intact and injured skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

## Contraindications

Do not use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

## Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of LBF Sterile No Sting Barrier Film Foam Applicator. May not be suitable for use on babies under 30 days old. Dispose of safely after use.

## Instructions For Use

1. Skin should be clean and dry prior to application.
2. Wear sterile gloves to maintain sterility when using this product.
3. To open the sachet, tear along either notch across the top.
4. Remove the foam applicator from the sachet and gently apply a uniform coating to the area of concern.
5. Dries in seconds. Allow LBF Sterile No Sting Barrier Film Foam Applicator to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.
6. If LBF Sterile is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.
7. When used under stoma pouches, adhesive tapes or dressings, reapplication is necessary with every change of adhesive appliance as the barrier is removed by the adhesive.
8. When used as a protectant against corrosive bodily fluids (including urine and faeces) and no adhesive products are applied to the skin, reapplication is recommended every 24-48 hours, depending on the frequency of cleaning. In extreme cases (e.g. constant diarrhoeal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
9. If desired, the film can be removed by using most medical adhesive removers.

## Ingredients

















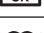

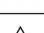

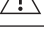


Siloxanes

## Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.

## Warning

LBF Sterile No Sting Barrier Film Foam Applicator is highly flammable. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

	Manufacturer		Single Sterile Barrier System		Do not use if packaging is damaged and consult instructions for use
	EC Representative		Single Sterile Barrier System with Protective Packaging Outside		Do Not Re-sterilise
	Catalogue Number		Sterilised by Irradiation		Do Not Re-use
	Batch Code		Use-by Date		Keep Dry
	Medical Device		Flammable		
	Unique Device Identifier		Sachet Material Not Yet Recycled		
	Country of Manufacture		Recycling (Paper and Cardboard)		
	Instructions for Use		Not Made with Natural Rubber Latex		
	Caution/Warning		Do Not Flush		
	Keep away from Sunlight				

Product Code	Description	No. Per Box
3824	LBF Sterile No Sting Barrier Film Foam Applicator, 1ml	5
3825	LBF Sterile No Sting Barrier Film Foam Applicator, 2ml	5

## CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.  
Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk  
CliniMed® and LBF® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd, Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022  
PID 10876

  
1639 0120