



CliniSure[®]
URINARY SHEATH



To check scale this line should be 10cm
(set print page size to 'actual size' or 100%)

1. The sizing guide should be used when your penis is flaccid.
2. Place the guide midway along your penis at the widest point and check the different sizes until you find one that fits snugly with no gaps.
3. If you measure between two sizes, you should choose the tighter option.

Please contact us on 0808 1596 017 for help with measuring if this is your first time using the CliniSure Sheath.

If you are unsure which size to use or need any help with sizing, please ask your healthcare professional.