

# Instillaquill®

## Prescribing Information:

**Composition:** Each 100g of Instillagel contains: Lidocaine Hydrochloride 2.0g, Chlorhexidine Digluconate 0.25g, Methyl Hydroxybenzoate 0.06g, Propyl Hydroxybenzoate 0.025g **Uses:** Catheterisation, cystoscopy, exploratory and intra-operative investigations, exchange of fistula catheters, protection against iatrogenic damage to the rectum and colon. Gynaecological investigations.

**Dosage and administration:** Unless otherwise prescribed by a doctor. Urethral catheterisation: instil 6-11ml of gel into the urethra. The anaesthetic effect begins after 3-5minutes.

**Contraindications, Warnings, Precautions and Interactions:** Instillagel® must not be used in patients with known hypersensitivity to the active ingredients or any of the excipients. It should not be used in patients who have damaged or bleeding mucous membranes. Use with caution in patients with impaired cardiac conditions, hepatic insufficiency and in epileptics. Difficulty in swallowing may occur with an increased risk of aspiration and biting trauma. Use with caution in patients receiving antiarrhythmic drugs.

**Undesirable effects:** In spite of the proven wide safety range of Instillagel®, undesirable effects of lidocaine are possible where there is severe injury to the mucosa; for example, anaphylaxis, fall in blood pressure, bradycardia or convulsions.

**Presentations:** Pre-filled disposable syringes; for single use only, 6ml and 11ml; packs of 10.

**NHS Price:** 10 x 6ml £10.50, 10 x 11ml £11.00.

**Legal Category:** Pharmacy P.

**Marketing Authorisation Number:** PL 03377/0002. **Marketing Authorisation Holder:** Farco-Pharma GmbH, Gereonsmuhlgasse 1-11, D-50670, Cologne, Germany.

**Further information is available from:** CliniMed Limited, Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks, HP10 9QY. Tel: 01628 850100 Email: info@clinimed.co.uk **Last revised:** January 2023

**Information about adverse event reporting can be found at [www.mhra.gov.uk/yellowcard](http://www.mhra.gov.uk/yellowcard). Adverse events should also be reported to Farco-Pharma on 0049221594061**

## Product Description

**Instillaquill® is a single use extension tube to fit an Instillagel® syringe. It is designed to provide 2% lidocaine gel to the cervical canal prior to gynaecological investigation procedures.**

The narrower distal part of Instillaquill® makes it suitable for use in both multiparous and nulliparous women, whilst the adjacent collar acts as a guide to the length of tube inserted into the cervical canal. Instillaquill® is indicated for use in day case surgery or minor out-patient procedures such as endometrial biopsy or out-patient hysteroscopy.

## Instillagel®

**Instillagel's anaesthetic, antiseptic and lubricating actions coupled with its unique syringe delivery system have made it the gel of choice in many UK hospitals.**


Widely used prior to male and female catheterisation Instillagel® has also allowed urological out-patient procedures – such as flexible cystoscopy – to be carried out with maximum efficiency and minimum patient discomfort.

## Ordering Information

Sterile / Single use only

Product	Reference	Pack quantity
Instillagel® 6ml	40-006	10 syringes
Instillagel® 11ml	40-011	10 syringes
Instillaquill®	40-200	10 quills

Note: It is recommended that following the insertion of Instillagel®, 5 minutes should be allowed to achieve optimum anaesthetic effect. Instillagel® is available in 6ml and 11ml presentations. 6ml Instillagel® is indicated for use in female gynaecological investigations. Instillaquill® is supplied sterile in a blister pack and should be discarded if the pack has been damaged.

 Instillaquill® and Instillagel® are marketed and distributed by: CliniMed Limited, Cavell House, Knaves Beech Way, Loudwater,

High Wycombe, Bucks. HP10 9QY

**Tel:** 01628 850100

**Email:** info@clinimed.co.uk

**Web:** www.clinimed.co.uk

**UK  
CA  
0120** **CE  
1639**

CliniMed® and Instillaquill® are registered trademarks of CliniMed (Holdings) Limited. © CliniMed Ltd 2023

Instillagel® is a product of Farco-Pharma GmbH, Cologne, Germany.



Advena Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013, Malta

PID 10839

**CliniMed®**

1

### Open blister pack where indicated.


Insert the Instillagel® nozzle into the Instillaquill® opening and remove from blister pack.



2

### Open vagina by use of a speculum and insert Instillaquill® into the cervical canal.


It may be necessary to apply an atraumatic forcep to the anterior lip of the cervix to locate the external cervical os in line with the vagina.

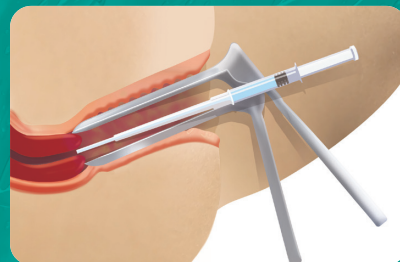

3

### Prime Instillaquill® with gel.


Introduce Instillaquill® to a point where the collar on the tube reaches the neck of the cervix.

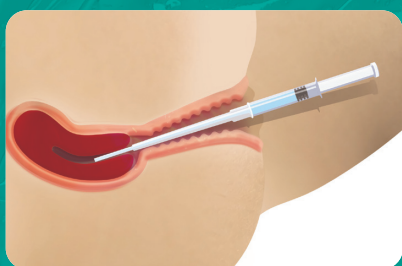

4

### Withdrawal.

Slowly withdraw Instillaquill® whilst at the same time injecting sufficient gel at a steady rate to coat the cervical canal. On completion remove Instillaquill® and dispose of together with the used Instillagel® syringe.

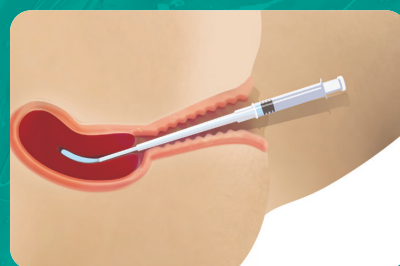

Symbol	Definition
	Manufacturer
	EC Representative
	Catalogue Number
	Batch Code
	Medical Device
	Unique Device Identifier
	Date of Manufacture/Country of Manufacture
	Instructions for Use
	Caution/Warning
	Keep away from Sunlight
	Do not use if packaging is damaged and consult instructions for use

Symbol	Definition
	Keep Dry
	Do not re-use
	Caution
	Expiry date
	Sterilised using Ethylene Oxide
	Single sterile barrier system
	Single Sterile Barrier System with protective packaging outside
	Not Made with Natural Rubber Latex
	Recycling (Paper)
	Recycling (Paper and Cardboard)
	Recycling (PVC Plastic or Vinyl)

CliniMed Ltd, Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks, HP10 9QY  
**Tel:** 01628 850100 **Email:** info@clinimed.co.uk **Web:** www.clinimed.co.uk  
 Last Revised: 2023

**UK CA** **CE**  
 0120 1639

**EC REP** Advena Ltd, Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013, Malta

PID 10839