



## Instructions

*The sizing guide should be used when your penis is flaccid.*

1. Place the guide midway along your penis at the widest point and check the different sizes until you find one that fits snugly with no gaps.
2. If you measure between two sizes, you should choose the tighter option.

Please contact us for help with measuring if this is your first time using the CliniSure Sheath. If you are unsure which size to use or need any help with sizing, please ask your healthcare professional.

**CliniMed®**

CliniMed Ltd. Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY.

Tel: 01628 850100 Email: [info@clinimed.co.uk](mailto:info@clinimed.co.uk) Web: [www.clinimed.co.uk](http://www.clinimed.co.uk)

CliniMed® and CliniSure® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed Ltd 2025.

PID 12349