



**CliniShield<sup>®</sup>**  
**ADVANCE**

## **Protective Barrier Cream**

CliniMed's cost-effective 'Clini' Range consists of CliniShield<sup>®</sup> Advance Barrier Film, CliniShield<sup>®</sup> Advance Protective Barrier Cream, CliniShield<sup>®</sup> Advance Protective Ointment, CliniShield<sup>®</sup> Advance 360° Foam & Spray Skin Cleanser and the CliniPeel<sup>®</sup> Medical Adhesive Removers.

# CliniShield<sup>®</sup> ADVANCE

## Protective Barrier Cream

---

### Product Description

CliniShield Advance Protective Barrier Cream provides an effective barrier to protect intact and injured skin from the harmful effects of bodily fluids from Moisture-Associated Skin Damage (MASD).

### Indications for use

CliniShield Advance Protective Barrier Cream offers waterproof, long-lasting protection (up to 24h) against moisture-associated skin damage (MASD) for both intact and injured skin. It is indicated for use in incontinence care, protecting skin from exposure to urine, faeces, and other bodily fluids. The cream moisturises and provides a protective barrier restoring the skin's hydration. The product may be used on adults and children.

### Benefits

- Cost effective: Moisturising formula covers a large area of skin
- Long lasting protection (up to 24h) from Moisture-Associated Skin Damage
- Can be used on intact and injured skin
- Waterproof, non-greasy formulation that resists wash-off, is easy to apply, and reduces the number of applications needed
- Does not block incontinence pad absorbency
- Skin friendly, non-sting and non-sticky formulation
- Dries rapidly on the skin
- Alcohol, fragrance, parabens and phthalates free. Not made with natural rubber latex
- Helps prevent skin irritation, breakdown, and damage caused by incontinence (urine and faeces)
- Suitable for use on children

### Contraindications

Do not apply to infected skin, open wounds, and deep puncture wounds. Do not use if known allergy or sensitivity.

### Caution

Use only as directed. For external use only. Do not use near eyes, ears, nose, or mouth. For single patient use. May not be suitable for use on babies under 30 days old. Dispose of safely after use.

### Instructions for use

1. Skin should be clean and dry prior to application.
2. Apply a thin, even layer to the area to be protected.
3. If CliniShield Advance Protective Barrier Cream is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.
4. Re-application is recommended every 24 hours, depending on the frequency of cleaning. For extreme cases of diarrhoea it might be necessary to reapply the barrier cream after each episode.

# CliniShield® ADVANCE

## Protective Barrier Cream

### Warning





















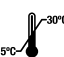
Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. In the event that the tube or sachet is received damaged before use, contact CliniMed. Recycle empty tubes and cartons in accordance with local regulations.

### Ingredients

Aqua, Glycerin, Pectin, Caprylyl Methicone, Trimethylmethoxysilicate, Dimethicone, Cetearyl Glucoside, Sorbitan Olivat, Cetearyl Alcohol, C12-15 Alkyl Benzoate, Phenoxyethanol, Chlorphenisin, Citric Acid

### Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity.  
For shelf life, refer to the expiry date on the tube or sachet.

	Manufacturer		Date of Manufacture/Country of Manufacture		Not Made with Natural Rubber Latex
	EU Representative				Recycling (Polypropylene) Cap
	Catalogue Number		Caution/Warning		Recycling (Low-density Polyethylene) Tube
	Batch Code		Keep Away from Sunlight		Recycling (Non-corrugated cardboard) Carton
	Medical Device		Single Use		Non-recyclable Sachet
	Unique Device Identifier		Keep Dry		Do not use if packaging is damaged and consult instructions for use
	Instructions for Use		Single Patient Multiple Use		
	Use-by Date		Temperature Limit		

Product Code	Description	No. Per Box
3833	CliniShield® ADVANCE Barrier Cream 2g	20
3834	CliniShield® ADVANCE Barrier Cream 28g	1
3836	CliniShield® ADVANCE Barrier Cream 90g	1

