



CliniShield[®]
ADVANCE

**No Sting
Barrier Film Wipes**

CliniShield[®] ADVANCE

No Sting Barrier Film Wipes

Product Description

CliniShield Advance No Sting Barrier Film Wipes contain a no sting silicone fluid that forms a transparent film on the skin and that acts as a protective barrier against damage from bodily fluids and waste and skin stripping caused by repeated application and/or removal of adhesives.

Indications for use

CliniShield Advance No Sting Barrier Film Wipes are indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

Contraindications

CliniShield Advance No Sting Barrier Film Wipes are not intended for direct use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of the product in accordance with local regulations.

Instructions for use

1. Skin should be clean and dry prior to application.
2. To open the sachet, tear along either notch across the top.
3. Remove the wipe from the sachet and gently apply a uniform coating to the area of concern.
4. Dries in seconds. Allow CliniShield Advance to dry completely before covering. Once dried, it will not affect any adhesion of further appliances.
5. Re-apply every 12-48 hours depending on the frequency of cleaning or with every change of adhesive appliance.
6. If desired, the film can be removed by using most medical adhesive removers.

CliniShield[®] ADVANCE

No Sting Barrier Film Wipes

Warning

Product is highly flammable. Keep away from heat, hot surfaces, sparks, open flames, sources of static discharge and other ignition sources. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number and contact your local competent authority. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

Ingredients

Siloxane.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity.
For shelf life, refer to the expiry date on each sachet.

	Manufacturer		Date of Manufacture/Country of Manufacture		Keep Dry
	EU Representative				Do not flush wipes
	Catalogue Number		Caution/Warning		Not Made with Natural Rubber Latex
	Batch Code		Use-by Date		Recycling (Paper and Cardboard)
	Medical Device		Flammable		Sachet Material Not Yet Recyclable
	Unique Device Identifier		Keep away from Sunlight		Do not use if packaging is damaged and consult instructions for use
	Instructions for Use		Do not re-use		

Product Code	Description	No. Per Box
3829	CliniShield Advance No Sting Barrier Film Wipes	30

CliniMed[®]

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed[®] and CliniShield[®] are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2026.

 Advena Ltd. Tower Business Centre, 2nd Floor,
Tower Street, Swatar, BKR 4013 Malta

Rev. 07/2026

PID 12934

CE UK
CA