

CliniSorb[®]

Activated Charcoal Dressing



CliniSorb[®]

Activated Charcoal Dressing

Product Description

CliniSorb is a sterile activated charcoal cloth which is sandwiched between layers of viscose rayon and coated with polyamide. CliniSorb can be used to control odour for a range of wounds such as, pressure ulcers, leg ulcers, diabetic foot and fungating wounds. It can be used in both moderately and lightly exuding wounds, the latter with the addition of a separate primary dressing to prevent adherence.

Indications for use

CliniSorb is indicated for the management of malodorous wounds and can be used as a primary or secondary dressing. A separate primary dressing may be required in dry or lightly exuding wounds.

Caution

Use only as directed. For external use only. May not be suitable for use on babies under 30 days old. Dressings are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. Product is sterile unless open or damaged. Dispose of the product in accordance with local regulations and clinical waste practices.

Instructions For Use

1. Carefully remove the dressing from the pouch using an aseptic technique. If necessary, you can cut the dressing to fit the wound shape.
2. Gently apply the CliniSorb Activated Charcoal Dressing to the wound. Either side of the CliniSorb Activated Charcoal Dressing can be placed on the wound. A separate primary dressing may be required in dry or lightly exuding wounds.
3. Cover and secure the CliniSorb Activated Charcoal Dressing in place using adhesive tape.
4. CliniSorb Activated Charcoal Dressing can remain on the wound for up to 7 days. A separate primary dressing may be required in dry or lightly exuding wounds.

CliniSorb®

Activated Charcoal Dressing

Warning

Keep out of reach of children. Do not use if the packaging is damaged and contact the legal manufacturer, CliniMed. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and in the event of serious incident occurring in relation to the use of this device, please report this to the manufacturer, quoting the product reference and LOT number on the device and contact your local competent authority.

Ingredients

Woven activated charcoal cloth, viscose-rayon polyamide coating.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each pouch

	Manufacturer		Single Sterile Barrier System with Protective Packaging Outside		Not suitable for recycling (film of pouch)
	EU Representative				Temperature Limit
	Catalogue Number		Sterilised by Irradiation		Not Made with Natural Rubber Latex
	Batch Code		Use-by Date		Keep Dry
	Medical Device		Do not use if packaging is damaged and consult instructions for use		Single Sterile Barrier System (single dressing only)
	Unique Device Identifier		Do Not Re-sterilise		
	Country of Manufacture		Do Not Re-use		
	Instructions for Use		Recycling (Paper)		
	Caution/Warning		Recycling (Paper and Cardboard)		
	Keep away from Sunlight				

Product Code	Description	No. Per Box
2305	CliniSorb 10 x 10cm	10
2310	CliniSorb 10 x 20cm	10
2315	CliniSorb 15 x 25cm	10

CliniMed®

 CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk
CliniMed® and CliniSorb® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2026.

 Advena Ltd, Tower Business Centre, 2nd Floor,
Tower Street, Swatara, BKR 4013 Malta

Rev. 07/2026
PID 12935


1639 0120