

LBBF®

STERILE
NO STING
BARRIER FILM WIPES

REF 3820

CliniMed®



Sterile No Sting Barrier Film Wipes

Product Description

LBF Sterile No Sting Barrier Film Wipes contain a no sting healthcare-grade silicone fluid that forms a transparent film on the skin and that acts as a protective barrier against damage from bodily fluids and waste and skin stripping caused by repeated application and/or removal of adhesives.

Indications for use

LBF Sterile No Sting Barrier Film Wipes are indicated for the protection of intact and injured skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

Contraindications

LBF Sterile No Sting Barrier Film Wipes are not intended for direct use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of the product in accordance with local regulations.

Instructions For Use

1. Skin should be clean and dry prior to application.
2. Wear sterile gloves to maintain sterility when using this product.
3. To open the sachet, tear along either notch across the top.
4. Remove the wipe from the sachet and gently apply a uniform coating to the area of concern.
5. Dries in seconds. Allow LBF Sterile to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.
6. If LBF Sterile is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.
7. When used under stoma pouches, adhesive tapes or dressings, reapplication is necessary with every change of adhesive appliance as the barrier is removed by the adhesive.
8. When used as a protectant against corrosive bodily fluids (including urine and faeces) and no adhesive products are applied to the skin, reapplication is recommended every 24-48 hours, depending on the frequency of cleaning. In extreme cases (e.g. constant diarrhoeal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
9. If desired, the film can be removed by using most medical adhesive removers.

Warning

Product is highly flammable. Keep away from heat, hot surfaces, sparks, open flames, sources of static discharge and other ignition sources. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number and contact your local competent authority. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

Ingredients

Siloxanes.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.

	Manufacturer		Single Sterile Barrier System		Keep away from Sunlight
	EU Representative		Single Sterile Barrier System with Protective Packaging Outside		Keep Dry
	Catalogue Number		Sterilised by Irradiation		Do not use if packaging is damaged and consult instructions for use
	Batch Code				
	Medical Device		Use-by Date		Do Not Re-sterilise
	Unique Device Identifier		Flammable		Do Not Re-use
	Country of Manufacture		Sachet Material Not Yet Recyclable		Do Not Flush
	Instructions for Use		Recycling (Paper and Cardboard)		
	Caution/Warning		Not Made with Natural Rubber Latex		

Product Code	Description	No. Per Box
3820	LBF Sterile No Sting Barrier Film Wipes	30