

Appeel® Sterile Medical Adhesive Remover Liquid Sachet, 5ml

Product Description

Appeel Sterile No Sting Medical Adhesive Remover Liquid Sachet contains a no sting healthcare-grade silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. Appeel Sterile No Sting Medical Adhesive Remover Liquid Sachet helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

Indications for use

Appeel Sterile No Sting Medical Adhesive Remover Liquid Sachet is indicated for use on intact and injured skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

Contraindications

Not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of safely after use.

Instructions for use

1. Skin should be clean and dry prior to application.
2. Wear sterile gloves to maintain sterility when using this product.
3. To open the sachet, pinch at the point indicated on the sachet and tear along the perforations across the top.
4. Holding the tear-off strip, pull upwards and sharply to remove the tear-off strip from the delivery applicator.
5. Continue to pinch the sachet at the pinch point. Tip the sachet towards the adhesive appliance and then gently release the pressure from the pinch point to control the flow of liquid.
6. Allow the liquid to drip over the adhesive appliance to release it from the skin.
7. Gently lift and repeat as needed. Use additional sachets if required for larger adhesive appliances.
8. Dries in seconds. Allow Appeel Sterile No Sting Medical Adhesive Remover Liquid Sachet to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.



Warning

Appeel Sterile is highly flammable. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged.

Ingredients: Siloxanes.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity.
For shelf life, refer to the expiry date on each sachet.

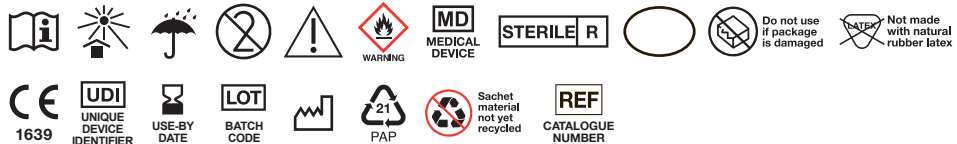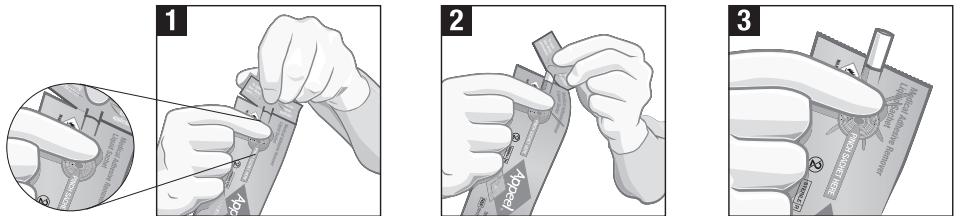

Product Code	Description	No. Per Box
3510-10	Appeel Sterile Medical Adhesive Remover Sachet, 5ml	10
3510-20	Appeel Sterile Medical Adhesive Remover Sachet, 5ml	20

Last revised November 2020

CliniMed®
CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
Tel 01628 850100 Fax 01628 527312 Web www.clinimed.co.uk
CliniMed® & Appeel® are registered trademarks of CliniMed (Holdings) Limited.
EC REP Advena Ltd, Tower Business Centre, 2nd Floor,
Tower Street, Swatara, BKR 4013 Malta

PID 9349