



**Morform[®] Motion Management
Super Absorbent Desiccant Sachets**

Morform[®]

Motion Management Super Absorbent Desiccant Sachets

Product Description

Morform is intended to gel the effluent within the ostomy pouch. The intended clinical benefits of Morform are to reduce leakage from ostomy pouches containing liquid output, which in turn reduces damage and discomfort to peristomal skin and reduces the frequency of ostomy pouch emptying making the ostomy pouch easier to manage. Morform also intends to reduce social embarrassment arising from pouch leakage, pouch noise and subsequent malodour. The intended users of Morform are persons with a faecal output stoma. Morform is compatible for use with all ostomy pouches.

Instructions for use

1. Place one sealed sachet into your pouch before fitting to your body.
2. Additional sachets may be inserted into drainable pouches after emptying.
3. Morform is non-toxic and flushes away in the usual manner.



Ingredients

Sodium polyacrylate

Storage and shelf life


Store away from direct sunlight at ambient temperature. Keep dry. For shelf life, refer to the expiry date on the pouch.

Caution

Use only as directed. For external use only. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the intended users. Dispose of outer sachet in a domestic rubbish bin. Risks associated with use have been reduced as far as practically possible. Sodium polyacrylate is not classified as a hazardous substance according to Regulation (EC) No 1272/2008. Spilled product coming into contact with water or moisture causes surfaces to become extremely slippery. The only interaction Morform has with its environment is dissolving in the faecal fluid of the ostomy collection pouch. Risks of undesirable side effects have been reduced as far as practically possible.

Warning

Keep out of reach of children. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Seek specialist advice where the device does not work as intended.

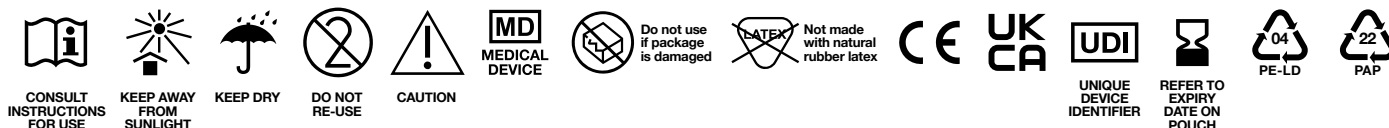

Product Code	Description	Quantity
3837	Morform® Motion Management Super Absorbent Desiccant Sachets	60 Sachets

Last revised 09/2021