

CliniSorb[®]

Activated Charcoal Dressing



REF 2305/2310/2315

CliniMed[®]

CliniSorb[®]

Activated Charcoal Dressing

Product Description

CliniSorb is a sterile activated charcoal cloth which is sandwiched between layers of viscose rayon and coated with polyamide. CliniSorb can be used to control odour for a range of wounds such as, pressure ulcers, leg ulcers, diabetic foot and fungating wounds. It can be used in both moderately and lightly exuding wounds, the latter with the addition of a separate primary dressing to prevent adherence.

Indications for use

CliniSorb is indicated for the management of malodorous wounds and can be used as a primary or secondary dressing. A separate primary dressing may be required in dry or lightly exuding wounds.

Caution

Use only as directed. For external use only. May not be suitable for use on babies under 30 days old. Dressings are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. Product is sterile unless open or damaged. Dispose of safely after use.

Instructions For Use

1. Carefully remove the dressing from the pouch using an aseptic technique. If necessary, you can cut the dressing to fit the wound shape.
2. Gently apply the CliniSorb Activated Charcoal Dressing to the wound. Either side of the CliniSorb Activated Charcoal Dressing can be placed on the wound. A separate primary dressing may be required in dry or lightly exuding wounds.
3. Cover and secure the CliniSorb Activated Charcoal Dressing in place using adhesive tape.
4. CliniSorb Activated Charcoal Dressing can remain on the wound for up to 7 days. A separate primary dressing may be required in dry or lightly exuding wounds.

CliniSorb[®]

Activated Charcoal Dressing

Warning

Keep out of reach of children. Do not use if the packaging is damaged. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and in the event of serious incident occurring in relation to the use of this device, please report this to the manufacturer, quoting the product reference and LOT number on the device.

Ingredients

Woven activated charcoal cloth, viscose-rayon polyamide coating.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date on each pouch



Product Code	Description	No. Per Box
2305	CliniSorb 10 x 10cm	10
2310	CliniSorb 10 x 20cm	10
2315	CliniSorb 15 x 25cm	10

Last revised February 2021



CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
 Tel 01628 850100 Fax 01628 527312 Web www.clinimed.co.uk
 CliniMed[®] and CliniSorb[®] are registered trademarks of CliniMed (Holdings) Limited.

EC REP Advena Ltd. Tower Business Centre, 2nd Floor,
 Tower Street, Swatar, BKR 4013 Malta

PID 9725