



**Morform® Motion Management
Super Absorbent Desiccant Sachets**

Morform®

Motion Management Super Absorbent Desiccant Sachets

Instructions For Use

To gel ostomy collection pouch contents:

1. Place one sealed sachet into your pouch before fitting to your body.
2. Additional sachets may be inserted into drainable pouches after emptying.
3. Morform is non-toxic and flushes away in the usual manner.



Ingredients

Sodium polyacrylate

Storage and shelf life

Store away from direct sunlight at ambient temperature. Keep dry.
For shelf life, refer to the expiry date on the pouch.

Morform®

Motion Management Super Absorbent Desiccant Sachets

Caution

Use only as directed. For external use only. Sachets are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. Dispose of the gelled contents in the toilet bowl and flush away. Dispose of the outer sachet in a domestic rubbish bin.

Warning

Keep out of reach of children. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged.



Product Code	Description	Quantity
3837	Morform® Motion Management Super Absorbent Desiccant Sachets	60 Sachets

Last revised 05/2021

CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
Tel 01628 850100 Fax 01628 527312 Web www.clinimed.co.uk
CliniMed® and Morform® are registered trademarks of CliniMed (Holdings) Limited.

EC REP Advena Ltd, Tower Business Centre, 2nd Floor,
Tower Street, Swatar, BKR 4013 Malta

PID 9902