

**CliniMed**<sup>®</sup>

**NON-WOVEN SWABS**

**Size 10cm x 10cm**

**Non-sterile**

**Quantity per pack: 100 4-ply swabs**

# CliniMed<sup>®</sup>

## Non-Woven Swabs

---

### **Indications for use**

For general clean procedures (including around stomas) on intact skin only.

### **Contraindications**

Not intended for packing in operating sites, or direct use on broken skin.

### **Ingredients**

Viscose (70%), polyester (30%)

### **Storage and shelf life**

Store away from direct sunlight at ambient temperature. Keep dry.

### **Caution**

Use only as directed. For external use only. Swabs are for single use only. Reuse of this product may affect the performance and create additional risks to the user. Dispose of safely after use.

### **Warning**

Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged.

# CliniMed®

## Non-Woven Swabs

---



KEEP AWAY FROM SUNLIGHT



KEEP DRY



DO NOT RE-USE



MEDICAL DEVICE



DO NOT FLUSH



Do not use if package is damaged



Not made with natural rubber latex



Product Code	Description	Quantity
3900	CliniMed Non-Woven Swabs 10cm x 10cm	100 4-ply swabs

Last revised 05/2021

# CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.  
Tel 01628 850100 Fax 01628 527312 Web [www.clinimed.co.uk](http://www.clinimed.co.uk)  
CliniMed® is a registered trademark of CliniMed (Holdings) Limited.



Advena Ltd. Tower Business Centre, 2nd Floor,  
Tower Street, Swatar, BKR 4013 Malta

PID 9903