



CliniPeel[®]

**No Sting Medical Adhesive
Remover Wipes**

REF 4503

CliniMed[®]

CliniPeel® No Sting

Medical Adhesive Remover Wipes

Product Description

CliniPeel No Sting Medical Adhesive Remover Wipes contain a no sting silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. CliniPeel No Sting Medical Adhesive Remover helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

Indications for use

CliniPeel No Sting Medical Adhesive Remover Wipes are indicated for use on intact skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

Contraindications

CliniPeel No Sting Medical Adhesive Remover Wipes are not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Do not use near and do not drip fluid into eyes, ears, nose, or mouth. Wipes are individually packaged for single use only. Reuse of this product may affect the performance and create additional risks to the user. May not be suitable for use on babies under 30 days old. Dispose of safely after use.

Instructions for use

1. Skin should be clean and dry prior to application.
2. To open the sachet, tear along either notch across the top.
3. Remove the wipe from the sachet and gently wipe over the adhesive appliance to release it from the skin.
4. Gently lift and repeat as needed. Use additional wipes if required for larger adhesive appliances.
5. Dries in seconds. Allow CliniPeel to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.

CliniPeel® No Sting

Medical Adhesive Remover Wipes

Warning

CliniPeel is highly flammable. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the packaging is damaged. Contact CliniMed in the event that the sachet is received damaged before use.

Ingredients

Siloxane.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity.

For shelf life, refer to the expiry date on each sachet.

	Manufacturer		Date of Manufacture/Country of Manufacture		Keep Dry
	EC Representative				Do not flush wipes
	Catalogue Number		Caution/Warning		Not Made with Natural Rubber Latex
	Batch Code		Use-by Date		Recycling (Paper and Cardboard)
	Medical Device		Flammable		Sachet Material Not Yet Recyclable
	Unique Device Identifier		Keep away from Sunlight		Do not use if packaging is damaged and consult instructions for use
	Instructions for Use		Do not re-use		

Product Code	Description	No. Per Box
4503	CliniPeel No Sting Medical Adhesive Remover Wipes	30

CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed® and CliniPeel® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2023.

 Advena Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 05/2023

PID 11138

