



CliniShield[®]

ADVANCE

No Sting
Barrier Film Spray

REF 3831

CliniMed[®]

CliniShield[®] ADVANCE

No Sting Barrier Film Spray

Product Description

CliniShield Advance No Sting Barrier Film Spray contains a no sting silicone fluid that forms a transparent film on the skin and that acts as a protective barrier against damage from bodily fluids and waste and skin stripping caused by repeated application and/or removal of adhesives.

Indications for use

CliniShield Advance No Sting Barrier Film Spray is indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

Contraindications

Not intended for direct use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Do not use near eyes, ears, nose or mouth. Avoid direct inhalation or use in enclosed spaces. May not be suitable for use on babies under 30 days old.

Instructions for use

1. Skin should be clean and dry prior to application.
2. Spray a uniform coating to the area of concern.
3. Dries in seconds. Allow CliniShield Advance to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.
4. Re-apply every 12-48 hours depending on the frequency of cleaning or with every change of adhesive appliance.
5. If desired, the film can be removed by using most medical adhesive removers.

CliniShield[®] ADVANCE

No Sting Barrier Film Spray

Warning














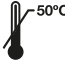




Flammable aerosol. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Do not expose to temperatures above 50°C/122°F. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the can is damaged. Contact CliniMed in the event that the can is received damaged before use. Recycle empty cans in accordance with local regulations.

Ingredients

Siloxane.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date at the base of the can.

	Manufacturer		Instructions for Use		Keep away from Sunlight
	EC Representative		Date of Manufacture/ Country of Manufacture		Single Patient Multiple use
	Catalogue Number				Recyclable (Aluminium)
	Batch Code		Caution/Warning		Not Made with Natural Rubber Latex
	Medical Device		Use-by Date		Upper Temperature Limit
	Unique Device Identifier		Flammable		Brimful Volume
					Phthalate Free


Product Code	Description
3831	CliniShield Advance No Sting Barrier Film Spray

CliniMed[®]

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed[®] and CliniShield[®] are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2023.

 Advena Ltd. Tower Business Centre, 2nd Floor,
Tower Street, Swatar, BKR 4013 Malta

Rev. 05/2023

PID 11140

