

#### Single Use Night Bag with Single Use T-tap. 2 Litre Instructions for use



# How to connect a single use night bag to a leg bag

The CliniSure<sup>®</sup> non-sterile night bag 2 litre is designed for single use only. The CliniSure non-sterile night bag 2 litre can be connected to a leg bag for overnight drainage or directly to a sheath for extra drainage capacity. Do not connect a non-sterile drainage bag directly to an indwelling catheter.

- 1. Wash and dry hands thoroughly.
- 2. Remove the CliniSure night bag from its packaging.
- **3.** Remove the pale blue protective cap from the night bag ridged connector.
- **4.** Insert the ridged connector securely into the leg bag tubing taking care not to twist.
- **5.** Attach the night bag to a stand. Ensure it is positioned lower than the bladder to allow effective drainage.
- 6. Open the outlet tap on the leg bag to allow continuous drainage.
- 7. Wash and dry hands thoroughly.



### How to empty a single use night bag

Ensure you empty your single use night bag before disposing of the bag.

- 1. Wash and dry hands thoroughly.
- 2. Close the outlet tap on the leg bag.
- 3. Disconnect the night bag from the leg bag tubing.
- **4.** Hold the outlet tap over a toilet or suitable receptacle for urine drainage.
- 5. Open the night bag outlet tap to allow the urine to empty.
- 6. Wash and dry hands thoroughly.

Important information: Connect to a leg bag or directly to an external continence management system. Do not connect directly to an indwelling urinary catheter. Do not reuse a non-sterile single use night bag. Ensure the night bag and leg bag if using are below the level of the bladder to allow drainage. Ensure the night bag is appropriately supported on a night bag stand or hanger.

**To dispose:** Dispose safely according to local guidelines, disposing of any urine into a toilet or suitable receptacle first. Do not flush the bag. Wash and dry hands thoroughly before and after touching drainage bags.

## **Further Information**



Product Code Description

CSSU2I

CliniSure non-sterile night drainage bag, 2 litre with single use T-tap

Quantity 10

#### **Using CliniSure?**

For free home delivery of your prescription continence supplies please contact SecuriCare<sup>®</sup>.



Call 0800 585 125 Extended opening hours Mon - Fri, 7.45am - 9pm Visit www.securicaremedical.co.uk



For further product information please contact CliniMed<sup>®</sup> Careline: 0800 036 0100 Email: info@clinimed.co.uk Web:www.clinimed.co.uk



VitalCare (Nanjing) Co., Ltd.

No. 86-8, Shuanggao Rd GEDZ Nanjing, 211316 PRC

CliniMed Ltd. Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY. Tel: 01628 850100 Fax 01628 527312 Email: enquiries@clinimed.co.uk Web www.clinimed.co.uk CliniMed®, CliniSure® and SecuriCare® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed Ltd 2020. PID 2908



VitalCare Limited. 6-9 Trinity Street, Dublin 2, Ireland