

Night Bags.
Lever & T-tap. 2 Litre
Instructions for use

**Clini**Med®


# Night Bags. Lever & T-tap. 2 Litre

#### How to connect a sterile night bag to a leg bag


For additional drainage overnight you can connect a CliniSure® sterile 2L reusable night bag to your leg bag at night-time. The CliniSure sterile 2L reusable night bags can also connect directly to an indwelling catheter or sheath for extra drainage capacity. If the night bag becomes disconnected from the indwelling catheter, the night bag cannot be reused and should be disposed of.

- 1. Wash and dry hands thoroughly.
- 2. Remove the CliniSure night bag from its packaging.
- **3.** Fill in the *Date Fitted* box that is on the night bag. This will allow you to monitor how long the night bag has been in use. Remember to change your night bag every 5-7 days.
- **4.** Remove the pale blue protective cap from the night bag ridged connector.
- **5.** Insert the ridged connector securely into the leg bag tubing, taking care not to twist.
- **6.** Attach the night bag to a stand using the hanger holes at the top of the night bag.
- **7.** Open the outlet tap on the leg bag to allow continuous drainage.
- 8. Wash and dry hands thoroughly.

#### T-tap operation

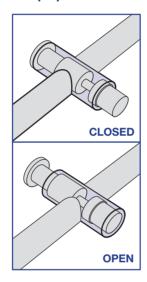

Lever tap operation

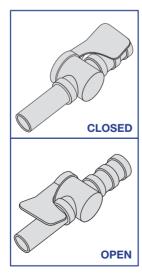

### How to connect a sterile night bag directly to an indwelling catheter or urinary sheath

- 1. Wash and dry hands thoroughly.
- 2. Remove the CliniSure night bag from its packaging.
- **3.** Ensure the outlet tap is in the closed position.
- **4.** Remove the pale blue protective cap from the night bag ridged inlet connector and avoid touching the connector.
- Fully insert the ridged inlet connector into the indwelling catheter or sheath, for a secure fit, taking care not to twist.
- **6.** Attach the night bag to a stand using the hanger holes at the top of the night bag. Ensure it is positioned lower than the bladder to allow effective drainage.
- 7. Ensure the tubing isn't kinked as this can restrict urine flow.
- 8. Wash and dry hands thoroughly.

#### How to empty a sterile night bag

- 1. Wash and dry hands thoroughly.
- 2. Hold the night bag outlet tap over a toilet or suitable receptacle for urine drainage.
- 3. Open the night bag outlet tap to allow the urine to flow from the bag.
- 4. Once emptied, fully close the night bag outlet tap.
- Dry the end of the outlet tap with a clean wipe or toilet tissue to remove any residual urine.
- 6. Wash and dry hands thoroughly.

## **Further Information**




Important information: Connect to a leg bag or directly to an indwelling urinary catheter or external continence management system. Change the night bag every 5-7 days. Do not reuse the night bag after disconnecting directly from a catheter due to increased infection risk and instead replace the night bag. Do not use the night bag if the packaging is damaged. Ensure the Date Fitted box is filled in to monitor how long the night bag has been in use. Ensure the night bag and leg bag if using are below the level of the bladder to allow drainage. Ensure the night bag is appropriately supported on a night bag stand or hanger. Empty the night bag regularly ensuring it does not overfill. After using the CliniSure night bag with a leg bag place the pale blue protective cap back on the ridged connector of the night bag and store at ambient room temperature.

**To dispose:** Empty the urine from the used night bag into the toilet or suitable receptacle and dispose of the used night bag according to local guidelines. Do not flush the bag. Wash and dry hands thoroughly before and after touching drainage bags.

Product Code	Description	Quantity
CSL2L	CliniSure sterile night drainage bag, 2 litre with lever tap	10
CST2L	CliniSure sterile night drainage bag, 2 litre with T-tap	10

#### **Using CliniSure?**

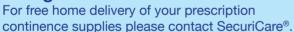



Call 0800 585 125 Extended opening hours Mon - Fri, 7.45am - 9pm Visit www.securicaremedical.co.uk





















For further product information please contact CliniMed® Careline: 0800 036 0100

Email: info@clinimed.co.uk Web:www.clinimed.co.uk



VitalCare (Nanjing) Co., Ltd.

No. 86-8, Shuanggao Rd GEDZ Nanjing, 211316 PRC

CliniMed Ltd. Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY. Tel: 01628 850100 Fax 01628 527312 Email: enquiries@clinimed.co.uk Web www.clinimed.co.uk CliniMed®, CliniSure® and SecuriCare® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed Ltd 2020. PID 9207



VitalCare Limited. 6-9 Trinity Street, Dublin 2, Ireland