

# Instillaquill®

Available for facilitating  
Instillagel® anaesthetic  
gel in gynaecological  
investigations.

Your **number**  
one choice for  
numbing pain.



**CliniMed®**

# What is Instillaquill®?

Instillaquill® is a sterile single use extension tube designed to fit an Instillagel® syringe. It is designed to provide local anaesthetic gel to the cervical canal prior to gynaecological investigation procedures.

Instillaquill® is indicated for use in day case surgery or minor outpatient procedures such as endometrial biopsy or outpatient hysteroscopy, where local anaesthesia may be required.

The narrower distal part of Instillaquill makes it suitable for use in multiparous and nulliparous women, whilst the adjacent collar acts as a guide to the length of the tube inserted into the cervical canal.

A clear plastic extension tube with a syringe-like plunger at one end. It is positioned diagonally across the frame, passing through a large circular graphic that has a dark blue outer ring and a light blue inner ring. The word 'Instillaquill®' is written in a teal font inside the circle.

Instillaquill®

Instillagel® is available in 6ml and 11ml presentations.

6ml Instillagel® is indicated for use in female gynaecological investigations.

Instillaquill® is supplied sterile in a blister pack and should be discarded if the pack has been damaged.

# Why should Instillagel® be your gel of choice?

Instillagel® is a triple action anaesthetic lubricant that has been widely used and trusted by our clinicians for over 30 years in the UK. It is the only licensed local anaesthetic and antibacterial gel available on the UK market and is a P licensed medicine, approved by the MHRA.

1

## Anaesthetic

**Contains 2% lidocaine**  
to provide local anaesthesia to the urethra and cervical canal

2

## Antiseptic

**Contains 0.25% chlorhexidine**  
providing broad-spectrum antiseptic properties

3

## Lubrication

**Acts as a lubricant**  
allowing smoother passage of Instruments

The use of Instillagel® to locally anaesthetise the cervix avoids the use of a hypodermic syringe and its inherent trauma.

## Instillagel®



# Instructions for use

**Note:** It is recommended that following the insertion of Instillagel®, 5 minutes should be allowed to achieve optimum anaesthetic effect.

1

## Open blister pack where indicated.


Insert the Instillagel® nozzle into the Instillaquill® opening and remove from blister pack.



2


## Open vagina by use of a speculum and insert Instillaquill® into the cervical canal.

It may be necessary to apply an atraumatic forcep to the anterior lip of the cervix to locate the external cervical os in line with the vagina.

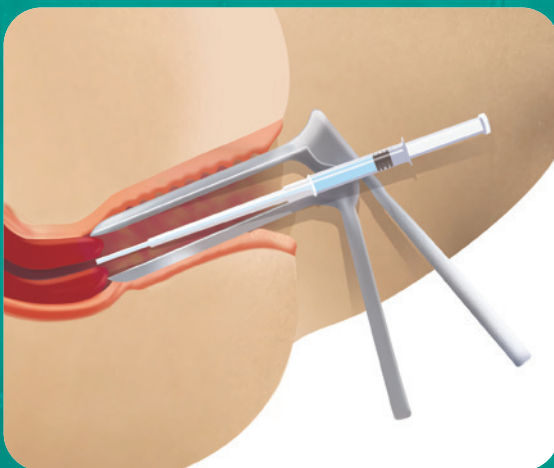


## **Prime Instillaquill® with gel.**

Introduce Instillaquill® to a point where the collar on the tube reaches the neck of the cervix.

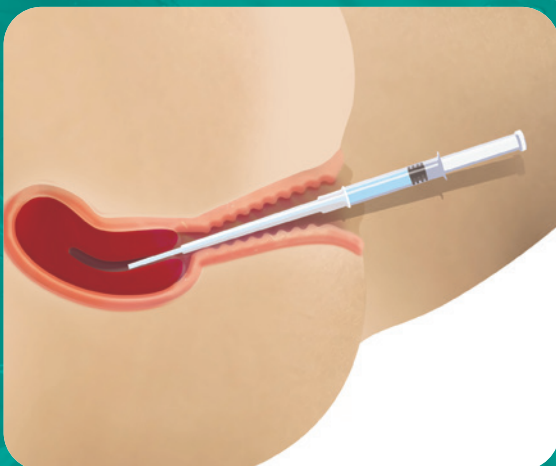

## **Withdrawal.**

Slowly withdraw Instillaquill® whilst at the same time injecting sufficient gel at a steady rate to coat the cervical canal. On completion remove Instillaquill® and dispose of together with the used Instillagel® syringe.

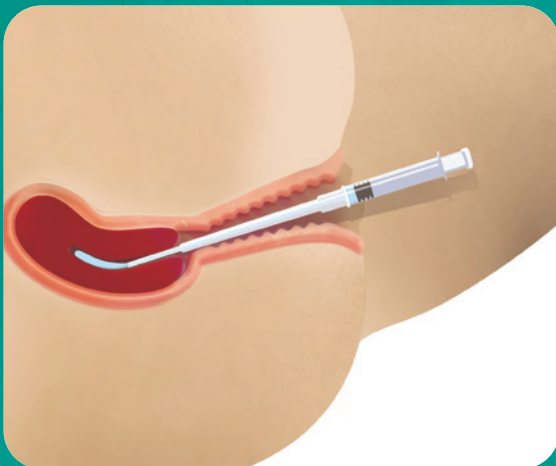



## Ordering Information

Sterile / Single use only

Product	Reference	Pack quantity
Instillagel® 6ml	40-006	10 syringes
Instillagel® 11ml	40-011	10 syringes
Instillaquill®	40-200	10 quills

**Prescribing Information:** **Composition:** Each 100g of Instillagel contains: Lidocaine Hydrochloride 2.0g, Chlorhexidine Digluconate 0.25g, Methyl Hydroxybenzoate 0.06g, Propyl Hydroxybenzoate 0.025g **Uses:** Catheterisation, cystoscopy, exploratory and intra-operative investigations, exchange of fistula catheters, protection against iatrogenic damage to the rectum and colon. Gynaecological investigations. **Dosage and administration:** Unless otherwise prescribed by a doctor. Urethral catheterisation: instil 6-11ml of gel into the urethra. The anaesthetic effect begins after 3-5 minutes. **Contraindications, Warnings, Precautions and Interactions:** Instillagel® must not be used in patients with known hypersensitivity to the active ingredients or any of the excipients. It should not be used in patients who have damaged or bleeding mucous membranes. Use with caution in patients with impaired cardiac conditions, hepatic insufficiency and in epileptics. Difficulty in swallowing may occur with an increased risk of aspiration and biting trauma. Use with caution in patients receiving antiarrhythmic drugs. **Undesirable effects:** In spite of the proven wide safety range of Instillagel®, undesirable effects of lidocaine are possible where there is severe injury to the mucosa; for example, anaphylaxis, fall in blood pressure, bradycardia or convulsions. **Presentations:** Pre-filled disposable syringes; for single use only. 6ml and 11ml; packs of 10. **NHS Price:** 10 x 6ml £10.50, 10 x 11ml £11.00. **Legal Category:** Pharmacy P. **Marketing Authorisation Number:** PL 03377/0002. **Marketing Authorisation Holder:** Farco-Pharma GmbH, Gereonsmühlengasse 1-11, D-50670, Cologne, Germany. **Further information is available from:** CliniMed Limited, Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks, HP10 9QY. Tel: 01628 850100 Email: [info@clinimed.co.uk](mailto:info@clinimed.co.uk) **Date:** March 2022

Information about adverse event reporting can be found at [www.mhra.gov.uk/yellow-card](http://www.mhra.gov.uk/yellow-card).  
Adverse events should also be reported to Farco-Pharma on 0049221594061

For more  
information:

Tel: **0800 316 0956**  
Email: **[info@clinimed.co.uk](mailto:info@clinimed.co.uk)**  
Web : **[www.clinimed.co.uk](http://www.clinimed.co.uk)**

# CliniMed®

CliniMed Ltd. Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY  
**Tel:** 01628 850100 **Fax:** 01628 527312 **Email:** [info@clinimed.co.uk](mailto:info@clinimed.co.uk) **Web:** [www.clinimed.co.uk](http://www.clinimed.co.uk)  
CliniMed Ltd., a company registered in England number 01646927  
Registered office: Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY  
Instillagel® is a registered trademark of Farco-Pharma GmbH and Instillaquill® and CliniMed® are registered trademarks of CliniMed (Holdings) Ltd. 2022 PID 10363